INTIMATE CARE

Soft Tampons 5er
Soft Tampons 10er

ART. NR.: 44320
ART. NR.: 44321

D

„SOFT TAMPONS“

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„SOFT TAMPONS“

INSTRUCTIONS

The stringless HOT INTIMATE CARE Soft Tampons allow for intercourse, sports, spa treatments, etc. during menstruation.

- Wash hands thoroughly before insertion or removal of HOT INTIMATE CARE Soft Tampons (fig.1).
- Remove the hygienically sealed Soft Tampon from its packaging.
- For easier insertion, apply a small amount of lubricant onto the Soft Tampon, or moisten it with clean water.
- The Soft Tampon can be easily inserted in a squatting position or when sitting on the toilet.
- Place the middle finger of one hand onto the middle of the Soft Tampon. Part the labia with the other hand and push the Soft Tampon into the vagina until it is in firm contact with the cervix (fig.2).
- The HOT INTIMATE CARE Soft Tampon can be worn for up to 8 hours (fig.3). Recommended time of use is 4 to 6 hours, depending on the flow of your period.
- Remove the Soft Tampon by pulling it out using your finger and thumb. If you have difficulty, we recommend that you remove the Soft Tampon in a squatting position for easier removal (fig.4).

IMPORTANT INFORMATION:

HOT INTIMATE CARE Soft Tampons is not a form of birth control. They are suitable to be used with condoms and spiral. Consult your doctor if you have recently had an spiral inserted (fig.5). Do not reuse Soft Tampons, as this can lead to illness and infections.

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